

Diversey™

TM

suma®

Med Super LpH



Low Alkaline Detergent

Description

Suma® Med Super LpH is a concentrated, low alkaline liquid washing and rinsing detergent specially formulated to provide excellent soil removal from medial instruments in soft to medium hard water conditions.

Features

Suma® Med Super LpH is a low alkaline liquid washing detergent which is suitable for use in a wide range of washer disinfector's and warewashing machines. The product is specially formulated for washing of medical instruments, anaesthesia equipment and suction bottles in washing and disinfecting machines. It is also excellent for cleaning of washbasins, kidney bowls and portatives in rinsing and disinfecting machines. The product is suitable for manual washing. It contains a blend of alkali and scale control agents, which will provide excellent soil removal and anti scaling in soft and medium hard water conditions.

Benefits

- Cuts through soil and dried on blood.
- Highly concentrated for greater economy in use.
- Low alkaline formulation protects instruments made of pH-sensitive materials.
- Ease of handling in convenient small pack.
- Wide neck bottle suitable for use with integral dosing pumps.
- Suma Med Super LpH Complies with recommendations of AS/NZS 4187:2014 as a instrument wash detergent.
- Class I Medical Device, AUST L 367453.

Application areas

- Healthcare



**Diversey**TM**suma**[®]**Med Super LpH**

Low Alkaline Detergent

Directions for use

Suma[®] Med Super LpH is normally applied using Diversey automatic dispensing equipment or integrated dosing pumps where fitted at a concentration of 1-3ml/L and at a temperature range of 50-60°C. Suma[®] Med Super LpH can also be dosed manually.

Consult your Diversey representative for optimum performance.

Rinse instruments thoroughly with potable water then proceed with appropriate disinfection or sterilisation process.

Technical information

Appearance	Clear colourless liquid	pH (neat)	7.0
Actives	Sequestrant blend	Shelf life	2 years
Specific Gravity	1.05	Storage	0-40°C
Dilution	1-3ml/L		

The above data is typical of normal production and should not be taken as a specification.

Product	Pack size	SKU Code	Availability
Suma [®] Med Super LpH	2 x 5L Wide Neck	5938057	AU
Suma [®] Med Super LpH	2 x 5L	3340633	AU

Safe handling and storage information

Full guidance on the handling and disposal of this product is provided in a separate Safety Data Sheet. Store in original container. Do not mix with other detergents or other chemicals. Store below 40°C.

Product compatibility

Under recommended conditions of use, Suma[®] Med Super LpH is suitable for use on most materials commonly encountered in hospital and CSSD.

Safety Reminder

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use, and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDSs are available online at www.diversey.com or by calling Toll free: Australia, 1800 647 779 or New Zealand, 0800 803 615.

For additional information, please contact us at:
Diversey Australia Pty Ltd Toll free: 1800 647 779, Diversey New Zealand Ltd Toll free: 0800 803 615 www.diversey.com

© 2018 Diversey, Inc. All Rights Reserved.

Sep-21